TEST BUDDY™ Control Solution

Instructions for Use

Intended Use: The Test Buddy™ Control Solution is a red liquid containing a known amount of glucose for use only with the Test Buddy Pet-Monitoring Blood Glucose Meter and Test Strips for dogs and cats to check that the meter and test strips are working together properly, and you are performing the test correctly.

Test Principle: Use control solution instead of blood to make sure your system is working the way it should. If the control test result displayed in the meter falls within the control test range printed on the test strip vial label for the level you are using, testing technique and system performance are acceptable.

Storage and Handling: Control solution must be stored upright, tightly sealed, at room temperature between 36°F–86°F. **DO NOT REFRIGERATE OR FREEZE.** Discard any bottle that appears to be cracked or leaking. **Write the date on the bottle label when you open a new control solution bottle. Discard control solution bottle either:**

- 3 months after the written date has passed (this is the open vial expiration date), OR
- after the expiration date printed on the label (EXP) has passed, whichever comes first.

Example only

Contents: Volume: 3 mL. **Chemical Composition:** Water, d-glucose, buffer, viscosity enhancing agents, salts, dye, and preservatives.

WARNING!

- For *in vitro* diagnostic use only. For use only with Test Buddy Pet-Monitoring Blood Glucose Test Strips and Meter for dogs and cats. **NOT FOR HUMAN USE.**
- Read all Instructions for Use (meter, test strips, control solution) before testing.
- Not for human or animal consumption. **Do not** drink or allow your pet to drink the control solution.
- **Do not** use the control test ranges printed on the test strip vial label as recommended blood glucose ranges for your dog or cat.
- **Do not** use test strips or meter for blood glucose testing if control test results are not within the control test range printed on the test strip vial label for the level you are using.


Important Information:

It is important to perform control tests with more than one level of control solution to ensure the system is working properly and your testing technique is good. Three levels of Test Buddy Control Solution are available (Level 1, 2, 3) that contain known amounts of glucose. Testing with at least 2 levels of control solution is recommended. For more information on obtaining different levels of control solution, call Customer Care at 1-800-803-6025, Mon.-Fri., 8AM-8PM EST.


Test Procedure:

1. Make sure meter, test strips, and control solution are at room temperature before testing.
2. Check expiration dates on the control solution bottle and test strip vial labels. **Do not** use expired control solution or test strips.
3. Wash hands and dry thoroughly.

4. Gently swirl or invert control solution bottle a few times to mix. **DO NOT SHAKE!**
5. Remove test strip from vial and recap immediately. Insert test strip into meter. The meter turns on and performs an automatic self-test. **Check that the correct animal type (dog or cat) symbol is displayed.**
6. Remove cap and turn control solution bottle upside down. Squeeze one drop of control solution onto a clean tissue. Wipe off bottle tip with the tissue.
7. Gently squeeze another drop of control solution onto a small piece of unused aluminum foil or clear plastic wrap. Recap control solution bottle tightly.
8. With test strip in meter, touch sample tip of test strip to the top of the control solution drop. **Do not** put control solution on top of the test strip. Wait for the control solution to be drawn into the test strip. Remove test strip from drop when meter begins testing.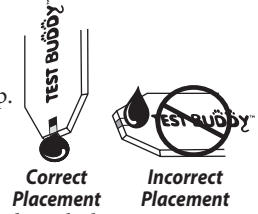

9. The result is displayed with the control symbol  when the test is complete. Compare the result with the control test range printed on test strip vial label for the level you are using.
10. Hold the meter with the test strip pointing down. Press the strip release button to discard the test strip into a suitable waste container.

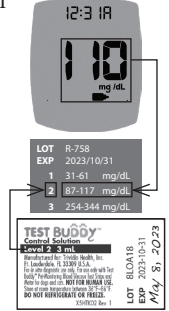Examples only

Expected Results: Control test results must be within the control test range printed on the test strip vial label for the control solution level you are using.

Troubleshooting Tips:

If the control test result is not within the range:

- Check the test strip and control solution expiration dates. **Do not** use expired test strips or control solution. If test strips and/or control solution have expired, discard them and repeat the test with a new test strip vial and/or control solution bottle.
- Check your testing technique. Repeat the control solution test with a new test strip.
- If the result is still outside the range, **do not** use the test strips and meter for testing blood. Contact Customer Care for assistance.
- If an error message appears, follow the instructions in the *Error Messages* section of the *Test Buddy Pet-Monitoring Blood Glucose System Owner's Booklet*.

WARNING! Do not use test strips or meter for blood glucose testing if control test results are not within control test range. See *Troubleshooting Tips* above or call Customer Care for assistance. **For animal health assistance, please contact a veterinarian.**

If you have any questions about the use of this product, please call **Customer Care toll free at 1-800-803-6025, Monday - Friday, 8AM-8PM EST;** or visit our website at www.testbuddymeter.com.

Manufactured for:
 **TRIVIDIA**
 HEALTH™
 FORT LAUDERDALE, FL 33309 U.S.A.

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